XUL002 - LED EXAMINATION LAMP
XUL003 - HALOGEN EXAMINATION LAMP

CHINESPORT[®]
ITALIA
REHABILITATION and MEDICAL EQUIPMENT

CE

XUL003 - HALOGEN EXAMINATION LAMP

cod. I0104AO

XUL002 - LED EXAMINATION LAMP

cod. I0104AOLED

The design of the light fixture is highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Precautions

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.
- **To change the bulb:**
 - Always disconnect the lamp from the power.
 - Remove the cover from the bulb (*image 1*).
 - Remove the bulb and screw in the new bulb.
 - Replace the rim cover (*image 2*).

Please contact your MIMSAL rep if you have any other problem.

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	XUL003 (I0104AO)	XUL002 (I0104AOLED)
Light bulb	Hal. Masterline ES 35W 8°	LED 7W 25°
Useful life	5.000 h	45.000 h
Lux at 50 cm	50.000 lux	15.000 Lux
Colour temperature	4.000° K	4.000° K
Colour Rendering Index	95%	80%
Electrical Protection Class	Class II	Class II
Length of Arm	72 cm	72 cm
Color	White	White
Protective glass	Borosilicate	Borosilicate
Power	220/230V	110/230V
Frequency	50/60 Hz	50/60 Hz
Fitting	Yes	Yes

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